Manual de usuario • User guidelines • Manual d'usuari
Manuel d'utilisation • Gebrauchsanweisung • Manuale d'uso
Gebruiksaanwijzing • Instrukcja obsługi • Manual de utilização
Brugsanvisning • Användarmanual • Brukerhåndbok
Käyttöopas • Notendahandbók • Lietotošanas pamācība
Ghidul utilizatorului • Priročnik za uporabo • Εγχειρίδιο λειτουργίας

HANDBIKE BATEC HIBRID

- HIBRID 2
- HIBRID 2 TETRA | QUAD HIBRID 2

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HANDBIKE BATEC HIBRID

- HIBRID 2
- HIBRID 2 TETRA

ÍNDICE

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- HIBRID 2
- QUAD HIBRID 2

CONTENTS

1. Intended purpose, indications and contraindications	4
2. Safety measures and general precautions	5
2.1. Before using your HANDBIKE BATEC HIBRID for the first time.....	5
2.2. During use.....	7
2.3. General precautions	9
2.4. Lifespan	10
3. Models: characteristics and parts	10
3.1. HANDBIKE BATEC HIBRID - HIBRID 2.....	11
4. Operating the HANDBIKE BATEC HIBRID / user guide	12
4.1. Delivery	12
4.2. Unpacking	12
4.3. Elevation, transport and storage	12
4.4. Attaching the HANDBIKE BATEC HIBRID	13
4.5. Detaching the HANDBIKE BATEC HIBRID.....	13
4.6. Using the HANDBIKE BATEC HIBRID 2 PARA controls	14
4.7. Using the HANDBIKE BATEC QUAD HIBRID 2 controls	15
4.8. Battery.....	17
4.9. Weights	23
5. Installation and adjustments	23
5.1. Installation of the HANDBIKE BATEC HIBRID on a wheelchair	23
5.2. User adjustments	23
6. Maintenance and repairs	25
7. Label	29
8. Warranty	30
8.1. Warranty period	30
8.2. Warranty conditions.....	30
8.3. Warranty card	31
9. Revisions list	32

1. INTENDED PURPOSE, INDICATIONS AND CONTRAINDICATIONS

The HANDBIKE BATEC HIBRID product turns a manual wheelchair into a hand-powered tricycle with electrical assistance. It allows people with disabilities who use a wheelchair to travel longer distances and more difficult routes than would be possible with a manual wheelchair alone. This makes them more independent in their daily lives and helps them get healthy physical exercise.

HANDBIKE BATEC HIBRID has CE marking as a class I medical device according to the requirements of Regulation (EU) 2017/745 concerning medical devices.

First, the handbike must be attached to a wheelchair. Once the two parts have been joined, we have a three-wheeled vehicle with the seat on the wheelchair and transmission and traction on the front wheel. To operate the handbike, just activate its transmission by pedalling with your hands or pressing on the accelerator. This will provide traction to the front wheel.

This product aims to provide wheelchair users (people with a disability that makes it impossible for them to walk) with attractive, practical products that are useful both in their daily life and for doing recreational activities and sport.

Thanks to the product configuration, users can do longer and more complicated trips than with a wheelchair alone, boosting independence and autonomy.

INDICATIONS

The HANDBIKE BATEC HIBRID is indicated for users with the following characteristics:

- With a disability affecting the lower limbs, who use a wheelchair and whose upper limbs are not affected and have full functionality of their hands. For these users, models with either the standard or PARA set-up are indicated.
- With a disability affecting the lower limbs, who use a wheelchair and whose upper limbs are partially affected and have good functionality of their arms and partial functionality of their hands, enough to accelerate, brake and turn without risk. For these users, models with the QUAD set-up are indicated. For those with spinal injuries, the most severe level that can use models with the QUAD set-up is functional C6. It is essential to have good biceps function and firm wrist extension.

Additionally, for safety reasons, the HANDBIKE BATEC HIBRID may only be used by people who:

- Have been instructed on how to operate it.
- Have the physical and psychological conditions required to operate it safely in all situations.
- Are able to brake the wheelchair and stop it if the device fails.

CONTRAINDICATIONS

The HANDBIKE BATEC HIBRID is not recommended in the following cases:

- People whose physical and/or psychological condition does not allow them to move around safely in situations with traffic.
- People who weigh over 110 kg.
- For the quad models, quadriplegic people who don't have at least the arm mobility described in this user manual.
- For the standard models, people who don't have enough hand strength to stop the device using the brake levers.
- For the hemi models, people who don't have enough strength in their working hand to stop the device using the brake levers.
- People with a visual impairment that impedes them from legally driving a vehicle.

If you are unsure, please check with a professional.

SPEED

The maximum speed of the product is over 6 km/h. In certain countries, this type of product is not allowed on public roads or cycle paths according to road traffic laws. As laws vary, it is your responsibility to be aware of the legal framework that applies in your specific country and whether it is compulsory to have insurance or a driving licence for a version that reaches over 6 km/h.

2. SAFETY MEASURES AND GENERAL PRECAUTIONS

2.1. BEFORE USING YOUR HANDBIKE BATEC HIBRID FOR THE FIRST TIME

IMPORTANT!

Before using your HANDBIKE BATEC HIBRID for the first time, read this user manual carefully.

- Don't exceed the maximum weight of 110 kg. This includes the user's weight and any other object being carried.
- The HANDBIKE BATEC HIBRID isn't compatible with all wheelchairs on the market. Before installing it, check the current BATEC Compatibility Table on our website (www.batec-mobility.com).

- It is important to start out slowly the first time you use your HANDBIKE BATEC HIBRID in order to learn the sensations you get and become familiar with it. Each new manoeuvre is a potential risk. Be particularly careful with turns and speed.
- Choose a calm place to use your HANDBIKE BATEC HIBRID the first time.
- Have someone help you become familiar with your HANDBIKE BATEC HIBRID. Each new manoeuvre can put you at risk of injury. Never use your HANDBIKE BATEC HIBRID alone until you have gained the necessary skill level. When someone is helping you, it is important for them to have read the user manual beforehand as well.
- The HANDBIKE BATEC HIBRID is a medical device classified as a technical aid for mobility and we recommend consulting your local regulations on the circulation of such vehicles. BATEC MOBILITY S.L. can provide you with the Declaration of Conformity as a medical device.
- Check local laws regarding add-on handbikes and wheelchairs, as they may vary depending on your location.
- Always do a safety check on your HANDBIKE BATEC HIBRID before each use. To do this, check:
 - Conditions of the fixed or removable axles on the rear wheels of your wheelchair.
 - The Velcro on the seat and backrest of your wheelchair.
 - Tyre pressure on your wheelchair. If it is too low, there is a greater risk of tipping over and greater wear (recommended pressure for wheelchair: see side of tyre).
 - HANDBIKE BATEC HIBRID tyre pressure (recommended pressure: depends on model).
 - Conditions of the HANDBIKE BATEC HIBRID frame.
 - All parts, in particular any screws, are properly tightened.
 - Brakes are working properly.
 - Make sure that all electrical connections are in good conditions and properly connected.
 - Make sure the battery is connected.
 - The HANDBIKE BATEC HIBRID attaches and detaches from the wheelchair properly.

CAREFUL! Before using

- Avoid damaging brake and gear-shift cables. The cables can get wrapped around the user if the crank arm isn't used properly.
 - When loading for transport, make sure the cables aren't pinned, pinched or damaged.
 - Do not use the HANDBIKE BATEC HIBRID if the cables are damaged.
-

2.2. DURING USE

- The HANDBIKE BATEC HIBRID must only be used to transport one person at a time.
- The HANDBIKE BATEC HIBRID is only recommended for people with the physical and mental state necessary to travel in situations with vehicle traffic.
- Avoid unnecessary risks like excess speed, fast turns and leaning to the side. Also avoid travelling over objects that may cause you to brake sharply or tip over.
- The safest way to drive the HANDBIKE BATEC HIBRID is on level surfaces without obstacles. Use ramps whenever possible. Always be extremely careful when crossing the road.
- Use reflectors on your HANDBIKE BATEC HIBRID and wheelchair or on your clothes to make you more visible to drivers. In situations with reduced visibility, always use lights.
- There is a risk of skidding on wet surfaces, gravel or irregular terrain, reduce your speed and use extra caution. Don't use the HANDBIKE BATEC HIBRID on mud or ice.
- Make sure your feet are properly positioned on the footrest of the wheelchair and that the seatbelt is properly adjusted.
- When you use your HANDBIKE BATEC HIBRID, control your speed and only ride as fast as appropriate for the given ground conditions. By maintaining a prudent speed, you will avoid unnecessary risks. Don't ride the HANDBIKE BATEC HIBRID faster than you would travel in a wheelchair alone.
- The maximum recommended speed on pavements and in pedestrian areas is 6 km/h.
- When you need to turn, remember to slow down and be extra careful. If you take a turn too quickly, you may lose control of the HANDBIKE BATEC HIBRID and run the risk of tipping over or crashing. Lean your body into the curve for better stability and to reduce the risk of tipping over.
- While driving and braking, hold the grips firmly with both hands in order to avoid any accidents caused by steering error.

CAREFUL!

- Going up or down kerbs isn't recommended, as there is a risk of tipping over. When you can't avoid a kerb, we recommend you:
 - Get help whenever possible.
 - Take the kerb perpendicularly, so that both wheels of your wheelchair go over the kerb at the same time. Going over a kerb in any other way significantly increases the risk of tipping over.

- Avoid banging the HANDBIKE BATEC HIBRID when going up or down over a kerb, as this could damage both the HANDBIKE BATEC HIBRID and your wheelchair.
 - The HANDBIKE BATEC HIBRID and the wheelchair must roll gently over the kerb on all wheels.
- See also figure 43. **(See fig. 43)**

-
- When travelling up or down a hill, we recommend you:
 - Get help whenever possible while on a hill.
 - Stay a straight course.
 - When going up steep hills, lean forward to put more weight on the front wheel and improve traction.
 - Avoid unnecessary turns and never turn sharply, as this puts you at risk of tipping over.
 - When going downhill, go slowly, braking continuously, and avoid braking sharply.
 - Don't use the brakes on your wheelchair; this could cause them to lock up and you to lose control.
 - The maximum recommended slope is 10°, as steeper inclines could make the parking brake less effective. Also take into account that the safe braking distance on hills can be much longer than on flat terrain.
 - When travelling long distances we recommend you carry an extra battery.
 - Make sure the grip on your wheels is proper conditions.
 - Make sure you are familiar with braking distances at various speeds.
 - Only use accessories approved by BATEC MOBILITY.
Check www.batec-mobility.com.
 - NEVER use your HANDBIKE BATEC HIBRID with a child on your lap, as there is a high risk
 - The BATEC ELECTRIC HANDBIKE could be affected by electromagnetic fields (for example, shop alarm systems, automatic doors, etc.) and its handling could also be affected by electromagnetic fields (for example, mobile phones, electric generators and high-voltage power sources).
 - The following manoeuvres are not allowed:
 - Riding up or down stairs or staircases.
 - Riding at an angle on the pavement.
 - 180° turns on steep hills.
 - Sharp movements.
 - Sharp braking on a curve.

 **CAREFUL!**

There is a risk that fingers may get smashed in several parts of the product (openings and spaces between fixed accessible parts) during intended use. (See fig. 1)

 **CAREFUL!**

There is a risk that feet may get smashed (openings and spaces between fixed accessible parts) during intended use. (See fig. 2)

 **CAREFUL! While using**

- Hold the handles with both hands firmly while driving and make sure the cables are facing up. Holding the crank arms in a horizontal or upward position could affect the cables.
 - Cables should never get folded or pinned, as this could damage and cause them not to work properly.
 - When you stop and release the handlebar of the HANDBIKE BATEC HIBRID, always make sure the crank arms are at their lowest point, meaning perpendicular to the ground.
-

2.3. GENERAL PRECAUTIONS

 **CAREFUL!**

- Remember to always turn off the BATEC ELECTRIC HANDBIKE before attaching or detaching it to the wheelchair.
- Do not use the BATEC ELECTRIC HANDBIKE if it behaves abnormally or erratically.
- Do not use the BATEC ELECTRIC HANDBIKE with dead batteries.
- Take your HANDBIKE BATEC HIBRID to a distributor or technical service provider authorised by BATEC MOBILITY regularly for servicing.
- Avoid contact or proximity to fire, particularly lit cigarettes.
- If the HANDBIKE BATEC HIBRID is exposed to direct sunlight or low temperatures for an extended period of time, the HANDBIKE BATEC HIBRID pieces may overheat (>41°C) or become too cold (<0 °C).
- Only use our charger to charge the battery.

- If you don't use your HANDBIKE BATEC HIBRID for an extended period of time, make sure the battery is fully charged and recharge it every 2 months to avoid damage caused by "deep discharge".
 - Keep the HANDBIKE BATEC HIBRID out of the reach of children when not under adult supervision.
 - Do not open the battery, as there is a risk of explosion and/or fire.
-

2.4. LIFESPAN

The BATEC ELECTRIC HANDBIKE has a lifespan of at least 5 years, as long as it is used according to the indicated operating conditions and receives proper maintenance and revisions, as indicated.

The life of the battery is between 200 and 500 charging cycles, and between 1 and 3 years, depending on several factors, including temperature, frequency of use, consistency of use and other individual issues (user weight, driving style, terrain, etc.).

DISPOSING OF THE PRODUCT

The BATEC ELECTRIC HANDBIKE is an electric/electronic device, so it must be disposed of properly. Please check with your local waste-management centre. Contact BATEC MOBILITY if you have any questions.

3. MODELS: CHARACTERISTICS AND PARTS

i INFORMATION!

The Batescs shown and described in this manual may not be exactly the same as the model you own. However, all of the instructions provided here apply. The manufacturer reserves the right to modify the weights, measurements and other technical details in this manual without prior notice. All figures, measurements and capacities in this manual are approximate and not technical specifications. The manufacturer reserves the right to modify this manual without prior notice. The most current version of the manual is always available online: <https://support.batec-mobility.com/en/support/>

There is only one model of the HANDBIKE BATEC HIBRID, named HIBRID 2, and it is designed for users with full use of their hands in its standard set-up. For users

without full use of their hands, we offer controls specially adapted to this situation, sold under the name QUAD.

i INFORMATION!

SKILLS NEEDED TO USE THE BATEC HANDBIKES: HANDBIKES BATEC are only recommended for people with the physical and psychological conditions to circulate in traffic. The standard HANDBIKE BATEC HIBRID models require full use of hands and arms.

To use the models with QUAD controls, users must have good arm function and partial hand function, enough to accelerate, brake and turn without danger. This is why BATEC MOBILITY always offers users the option to test the product before purchasing.

The most serious spinal cord injury compatible with use of the HANDBIKE BATEC HIBRID QUAD is functional C6 (good biceps function and wrist motion is essential). (See fig. 3)

3.1. HANDBIKE BATEC HIBRID - HIBRID 2

3.1.1. CHARACTERISTICS

HIBRID 2

- Material: Aluminium 7005 T6 / Aluminium 6061 T6
- Motor: Brushless 900 W 250 RPM / Peak power: 1050 W
- Top speed: 25 km/h.
- Battery: several battery options available:
 - 280 Wh (7.8 Ah) battery with 27*-40** km battery life
 - 522 Wh (14.5 Ah) battery with 50*-75** km battery life
 - 576 Wh (16 Ah) battery with 55*-80** km battery life

*Battery life calculated under the following conditions: 75-kg user, extra-light wheelchair, flat terrain, no wind and steady speed of 15 km/h. May vary $\pm 20\%$ depending on the battery.

**Average battery life combining arms and motor. Depends on the level of electrical assistance and strength of the user's arms.

- Super-fast 4A charger
- The HANDBIKE BATEC HIBRID power and steering system has a V-0 flammability rating, according to UL94 standards.
- Tyre: 20" ultra-grip

- Rim: 20" double-wall aluminium
- Brakes: Dual mechanical brake callipers with 200/203 mm disc brake with aluminium levers and parking brake.
- Headset: 1.1/8 black
- Handlebar: aluminium crank arms
- HANDBIKE BATEC ELECTRIC EASY-FIT regulation system (Pat. pend.)
- HANDBIKE BATEC ELECTRIC EASY-FIX anchor system (Pat. pend.)

3.1.2. PARTS OF THE HANDBIKE BATEC HIBRID - HIBRID 2

(See fig. 4, 5, 6, 7, 8, 9)

3.1.3. Parts of the HANDBIKE BATEC QUAD HIBRID - HIBRID 2

(See fig. 4, 5, 6, 8, 9, 10)

4. OPERATING THE HANDBIKE BATEC HIBRID / USER GUIDE

4.1. DELIVERY

Your new HANDBIKE BATEC HIBRID will be delivered in a cardboard box. In order to prevent damage during transport, parts that must be assembled later, like the battery, are packaged separately inside the cardboard box.

4.2. UNPACKING

- Check that the outside of the cardboard box hasn't been damaged in any way during transport.
- Remove all of the packing materials.
- Remove the parts of the HANDBIKE BATEC HIBRID (including the battery and charger) from the cardboard box and check that they are all there.
- Check that all the parts are in proper conditions, without any sort of damage, scratches, tears, dents, irregularities or other defects.
- If you find any problems, or if any pieces are missing, notify the authorised distributor immediately.

4.3. ELEVATION, TRANSPORT AND STORAGE

The HANDBIKE BATEC HIBRID has been designed so that users can easily remove and detach the battery on their own.

- When taking the HANDBIKE BATEC HIBRID on any means of transport (car, bus, train, plane, etc.), we recommend removing the battery before your journey: turn the battery key to OPEN, remove the key and then remove the battery.
- When lifting the HANDBIKE BATEC HIBRID, we recommend removing the weight as indicated in section 3.10 and the battery as per section 4.9.1.
- When transporting the HANDBIKE BATEC HIBRID in the boot of a vehicle, we recommend leaving it attached to a wheelchair whenever possible.

The BATEC HANDBIKE is not designed to be used as a means of transport inside a vehicle.

The HANDBIKE BATEC HIBRID ELECTRIC should be stored in a cool, dry place, protected from direct sunlight and rain. It should not be kept out of use for more than two months.

4.4. ATTACHING THE HANDBIKE BATEC HIBRID:

- Step 1: Get in front of the HANDBIKE BATEC HIBRID, with the handlebar centred in front of the wheelchair, with the brake on. **(See fig. 11, 12)**
- Step 2: Grab the HANDBIKE BATEC HIBRID by the handlebar and pull it towards you, rolling the frame stand until the detaching ramp on the anchoring bar touches the front axle of the bar on the wheelchair. **(See fig. 13)**
- Step 3: Tip the handlebar, lifting the front wheel, while pulling the HANDBIKE BATEC HIBRID towards you. When the detaching ramp has passed the front axle of the bar on the wheelchair, let go of the handlebar and put the front wheel back on the ground. **(See fig. 14, 15)**
- Step 4: Take off the brake on the wheelchair and push gently on the HANDBIKE BATEC HIBRID handlebar to make sure the front axle is securely in place.
- Step 5: Push the handle until it clicks, so that both the wheels on the HANDBIKE BATEC HIBRID stand frame and the front wheels on the wheelchair are off the ground. Your HANDBIKE BATEC HIBRID is now attached. **(See fig. 16)**

4.5. DETACHING THE HANDBIKE BATEC HIBRID

- Step 1: Make sure your HANDBIKE BATEC HIBRID is switched off. **(See fig. 17)**
- Step 2: Push the anchor lever forward with the palm of your hand until you feel it come free. Don't let go of the lever and set the HANDBIKE BATEC HIBRID on the ground. Advice: lean your torso back to take weight off the lever. This will make it much easier to detach the HANDBIKE BATEC HIBRID. **(See fig. 18)**
- Step 3: Tilt the handlebar, lifting the front wheel, while you push the HANDBIKE BATEC HIBRID away from you. **(See fig. 19)**

- Step 4: When the detaching ramp has gone over the front axle of the wheelchair bar, let go of the handlebar and set the front wheel down again. Your HANDBIKE BATEC HIBRID is now detached. (See fig. 20)

 **ADVICE!**

Practice attaching and detaching your HANDBIKE BATEC HIBRID calmly and you'll see that, in just a few weeks of use, it will become automatic.

4.6. USING THE HANDBIKE BATEC HIBRID 2 PARA CONTROLS

- You can find out the meaning of all the symbols used on the controls of the BATEC ELECTRIC HANDBIKE Figure 22. (See fig. 22)
- Turn on: Turn the key on the right side of the battery clockwise to the ON position. It will do a system check: don't start using your HANDBIKE BATEC HIBRID until it has finished with the system check and all the lights are on. (See fig. 17)

 **IMPORTANT!**

For the safety check to be completed properly, the wheel of the HANDBIKE BATEC HIBRID must be stopped and the user must have squeezed the brake levers on the HANDBIKE BATEC HIBRID at least once.

- Turn off: Turn the key on the battery anti-clockwise to the OFF position. If you leave it in the OPEN position, your HANDBIKE BATEC HIBRID will be off and the battery will be ready to remove. (See fig. 17)

There are two operating modes: Assisted manual propulsion and electric mode. In the first mode, the user propels the handbike using the crank arms and the motor provides assistance. The second mode is an emergency mode, only for situations when the user needs to use the handbike without turning the crank arms due to stability issues.

ASSISTED MANUAL PROPULSION

In Assisted manual propulsion mode, there are 3 levels of help (position 1, position 2 and position 3), each with a light indicator on the display panel. Level 1 provides the least help and level 3, the most. Users can change between the three levels using the buttons on the right handle. When the HANDBIKE BATEC HIBRID is switched on, it will always be in assisted manual propulsion mode, position 1. The button with the up arrow increases the level and the button with the down arrow is to switch to a lower level.

There is also a 9-speed gear train and gears can be selected using the levers on the right grip.

Using Assisted manual propulsion mode

Hold the grip firmly and turn the Batec crank arms forward, always with the cables up. When the sensor in the axis of the pedals detects the force, it will activate the electric motor. Use the buttons to change the level of assistance and use the gear-shift levers to change gears.

ELECTRIC PROPULSION MODE

To switch to electric mode, press the down arrow while in position 1. In electric mode, there is only one speed and one light indicator on the display panel.

Using Electric propulsion mode

The user controls the speed of the motor with the trigger on the electric grip. In this mode, the sensor on the pedals is not activated.

BRAKE

The HANDBIKE BATEC HIBRID has two brake levers on the handlebar to activate the disc brake. Pull on the lever to activate the brake.

TIP

Always keep two fingers on the brake lever so you can react quickly.

Pressing on the brake lever and the red lever at the same time blocks the brake lever, acting as a parking brake. (See fig. 21)

LIGHTS

There is a button on the HANDBIKE BATEC HIBRID handlebar to switch on the front and rear lights. When you push the button, the front light will go on in normal mode and the light indicator on the display panel will turn green. When you push the front light button again, they change to high-beam mode, and so on. To turn off the lights, hold the button down for 2 seconds. The front light is below the battery and the two rear lights, on either side of the stand frame. (See fig. 23, 24)

4.7. USING THE HANDBIKE BATEC QUAD HIBRID 2 CONTROLS

There are two operating modes: Assisted manual propulsion and electric mode. In the first mode, the user propels the handbike using the crank arms and the motor provides assistance. The second mode is an emergency mode, only for

situations when the user needs to use the handbike without mechanical traction using the MAGIC LEVER.

ASSISTED MANUAL PROPULSION

In Assisted manual propulsion mode, there are 3 levels of help (position 1, position 2 and position 3), each with a light indicator on the display panel. Level 1 provides the least help and level 3, the most. Users can change between the three levels using the buttons on the right handle. When the HANDBIKE BATEC HIBRID is switched on, it will always be in assisted manual propulsion mode, position 1. The button with the up arrow increases the level and the button with the down arrow is to switch to a lower level.

There is also a 8-speed gear train and gears can be selected using the levers on the right grip.

Using Assisted manual propulsion mode

Hold the grip firmly and turn the Batec crank arms forward, always with the cables up. When the sensor in the axis of the pedals detects the force, it will activate the electric motor. Use the buttons to change the level of assistance and use BATEC SHIFT LEVER.

MAGIC LEVER

Normal use

1. Put the MAGIC LEVER in position - B. (See fig. 44)
2. Push the pedal forwards. (See fig. 45)

If you can't start the handbike, use the COMBINED START

1. Hold the MAGIC LEVER in position - C. (See fig. 46)
2. Push the pedal 1/16 of a turn forwards. (See fig. 47)
3. Release the MAGIC LEVER. (See fig. 48)
4. Push the pedal forwards.

Using No resistance mode

1. Put the MAGIC LEVER in position - A. (See fig. 49)
2. Push the pedal forwards. (See fig. 50)

ELECTRIC PROPULSION MODE

To switch to electric mode, press the down arrow while in position 1. In electric mode, there is only one speed and one light indicator on the display panel.

Using Electric propulsion mode

The user controls the rotational speed of the motor by freely turning the pedals, in the forward direction, using the MAGIC LEVER.

BACK PEDAL BRAKE ON THE HANDBIKE BATEC HIBRID QUAD

- How it works: Turn the pedals backwards to brake. (See fig. 51)

BATEC QUAD GRIPS ON THE HANDBIKE BATEC HIBRID QUAD

User adjustments

1. Adjust wrist vertically. Tighten screw to 20 Nm. (See fig. 52)
2. Adjust wrist horizontally. Tighten screw to 20 Nm. 2. (See fig. 53)

BATEC SHIFT LEVER GEAR SHIFT FOR QUADRIPELGICS ON THE HANDBIKE BATEC HIBRID QUAD

User adjustments

1. Turn the pedal to find the point of maximum effort pressing inward (normally with the crank arm parallel to the floor). (See fig. 54)
2. Loosen the screw from the guide ring. (See fig. 55)
3. Put hand on the grip and align the slit on the guide ring with the centre of the arm. (See fig. 56)
4. Tighten the screw on the guide ring to 2 Nm. (See fig. 57)

How it works

1. The left hand presses inward. (See fig. 58)
2. Twist forward for shorter gears. (See fig. 59)
3. Twist backward for longer gears. (See fig. 59)

LIGHTS

There is a button on the HANDBIKE BATEC HIBRID QUAD handlebar to switch on the front and rear lights. When you push the button, the front light will go on in normal mode and the light indicator on the display panel will turn green. When you push the front light button again, they change to high-beam mode, and so on. To turn off the lights, hold the button down for 2 seconds. The front light is below the battery and the two rear lights, on either side of the stand frame. (See fig. 23, 24)

4.8. BATTERY

BATEC MOBILITY offers several 36-V lithium-ion batteries, 280 Wh and 522 Wh, all without memory effect and with a cable-free connection system.

THE 280 WH BATTERY COMPLIES WITH IATA-DGR REGULATIONS for air travel with batteries in mobility-aid devices.

Lithium ion batteries have many advantages over other batteries, mainly due to their flexibility. They don't have to be completely charged every time they are used, nor do they have to be used continuously or fully drained each time.

BATEC MOBILITY only uses batteries with the safest technology available, including the BMS smart battery management system.

HANDBIKE BATEC HIBRID batteries don't require any maintenance.

4.8.1 BASIC BATTERY USE

Inserting and removing the HANDBIKE BATEC HIBRID battery

You don't need to touch any connectors to insert or remove the battery from your HANDBIKE BATEC HIBRID, as this is done automatically. (See fig. 29)

- To remove the battery: Turn the key to the REMOVE BATTERY position (OPEN). Grab the handle of the battery and pull up gently until the battery comes fully of the guide. (See fig. 27)
- To insert: Grab the battery by the handle. Set the lower right corner of the battery in the side support tab until it clicks into the guide. Slide the battery down slowly until it is resting completely in the mount. (See fig. 28)

.....

IMPORTANT!

If you leave the battery in the OFF AND REMOVE KEY WITH DETACHING LOCK position (OFF), you won't be able to remove it.

.....

Connecting the battery:

To connect the HANDBIKE BATEC HIBRID battery, simply turn the key to the ON position and use your HANDBIKE BATEC HIBRID normally. (See fig. 17)

The battery charging system is protected by the smart battery management system (BMS), which cuts power if there is a surge or if any of the battery cells are too discharged.

.....

CAREFUL!

Never, under any circumstances, use the discharging port on the battery to charge it or power it in any way, whether with an external power source or regenerative brake system.

.....

Charging the battery:

- The HANDBIKE BATEC HIBRID battery is charged with the 3-pin XLR port on the side of the battery. (See fig. 30)

- Only charge the battery with the original BATEC MOBILITY charger provided with your HANDBIKE BATEC HIBRID.

CAREFUL!

Never, under any circumstances, use chargers from other manufacturers or a charger from another HANDBIKE BATEC HIBRID, as the specifications for each HANDBIKE BATEC HIBRID series are different.

- Recharge time for the BATEC MOBILITY battery is 3 hours. Both the charger and the battery management system cut the power when the battery is fully charged.
- Charging the battery: To charge the battery, the key must be in the OFF or OPEN position. You can charge the battery whether or not it is attached to the HANDBIKE BATEC HIBRID. Turn the cover of the charging port, plug the charger first into the port and then into a socket, then turn the charger on.

Storing the battery

Lithium ion batteries have a smart battery management system (BMS) that uses a small amount of power to monitor the battery at all times. The amount used is very small but if you store the battery fully discharged, in just a few weeks the BMS can run out of power.

If this happens, the battery can never be used again. As a result, we strongly recommend you follow these instructions carefully:

CAREFUL!

- Charge the battery fully before storing it.
- Always store the battery with the key in the OFF position.
- Charge the battery fully at least every 2 months.
- If the battery doesn't work after being stored for a period of time, for your safety it must be disposed of.

4.8.2. BATTERY SAFETY INFORMATION

It is widely known that lithium ion batteries can be dangerous, and for good reason. Batteries used in vehicles, like those in the Batecs, have enough stored energy and potentially dangerous material to cause a significant incident if not used properly or in the case of negligence.

BATEC MOBILITY ONLY USES LITHIUM BATTERY TECHNOLOGY IT CONSIDERS STABLE AND SAFE.

All BATEC MOBILITY batteries use safety systems, including the smart battery management system (BMS), which monitors the battery at all times.

In addition to all of the built-in safety systems, to minimise the danger if a problem were to arise with elements outside the battery, such as improper functioning of the electrical installations or problems with the general power grid, we strongly recommend you follow these instructions carefully:

CAREFUL!

- Only charge the battery under supervision.
- Don't charge the battery while using your BATEC ELECTRIC HANDBIKE: Danger of electrocution and accidents.
- Don't charge the battery while asleep at night.
- Charge the battery well away from any flammable objects.
- Never use the discharging port to charge the battery and vice versa, never use the charging port to discharge it.
- Only use the BATEC MOBILITY charger provided with your HANDBIKE BATEC HIBRID to charge the battery.
- Never charge the battery near a heat source.
- Never charge the battery in temperatures below 0°C.
- Always store the battery fully charged.
- Don't allow the inside of the battery to be manipulated in any way, as there is a risk of explosion and/or fire.

4.8.3. ADDITIONAL INFORMATION ON THE BATTERY

Using the battery in cold conditions

The performance of lithium batteries is limited, both in terms of power and battery life, in low temperatures (below 0°C). It isn't a dangerous situation but it is bothersome, so we recommend storing the battery inside under these circumstances. Never charge the battery in temperatures below 0°C.

Using the battery in damp conditions

The casing of the BATEC MOBILITY battery provides good protection against rain and dampness for the cells and circuits inside. This protection is enough for the battery to withstand these conditions occasionally but regular, prolonged exposure to damp and rainy conditions may cause the battery to stop working prop-

erly. Thus, we recommend you avoid these circumstances and use the HANDBIKE BATEC HIBRID as little as possible in rainy or snowy conditions.

Battery life

The life of the battery is between 200 and 500 charging cycles, and between 1 and 3 years, depending on several factors, including temperature, frequency of use, consistency of use and other individual issues (user weight, driving style, terrain, etc.).

To get the most out of your battery we recommend you:

IMPORTANT!

- Drive gently at moderate speeds.
- Avoid exposure to high temperatures.
- Avoid exposure to low temperatures (below 0°C).
- Avoid leaving the battery unused for long periods of time.

The battery won't just die suddenly; it gradually begins to perform more poorly. If your battery suddenly stops working, it is normally due to a technical problem and not the end of its lifespan.

Disposing of the battery

As with all batteries, the BATEC MOBILITY batteries must be disposed of properly. Check with your local waste-management centre. Contact BATEC MOBILITY if you have any doubts.

Battery indicator operation

The battery status indicator is on the display (LCD panel) of the BATEC ELECTRIC HANDBIKE. The indicator is made up of several bars that indicate the % of energy available. (See fig. 42)

IMPORTANT!

When the system detects that there is less than 20% battery charge available, it can limit the HANDBIKE BATEC HIBRID to position 1 (maximum speed: 6 km/h or 3 km/h, depending on the model) to save power.

4.8.4. TECHNICAL SPECIFICATIONS OF THE 280 WH BATTERY

- Dimensions: 350 x 140 x 60 mm
- Weight: 2.6 kg

- Power: 280 Wh
- Voltage: 36 V
- Maximum discharge current: 29 A
- Maximum charge current: 5 A
- Internal resistance: 200 mOhm
- BMS consumption: < 0.2 mA when on; < 0.02 mA when off
- HANDBIKE BATEC HIBRID autonomy:
 - HIBRID 2: 27 - 40 km

4.8.5. TECHNICAL SPECIFICATIONS OF THE 522 WH BATTERY

- Dimensions: 350 x 140 x 60 mm
- Weight: 4 kg
- Power: 522 Wh
- Voltage: 36 V
- Maximum discharge current: 29 A
- Maximum charge current: 5 A
- Internal resistance: 200 mOhm
- BMS consumption: < 0.2 mA when on; < 0.02 mA when off
- HANDBIKE BATEC HIBRID autonomy:
 - HIBRID 2: 50 – 75 km

4.8.6. TECHNICAL SPECIFICATIONS OF THE 576 WH BATTERY

- Dimensions: 350 x 140 x 60 mm
- Weight: 3.9 kg
- Power: 576 Wh
- Voltage: 36 V
- Maximum discharge current: 29 A
- Maximum charge current: 5 A
- Internal resistance: 200 mOhm
- BMS consumption: < 0.2 mA when on; < 0.02 mA when off
- HANDBIKE BATEC HIBRID autonomy:
 - HIBRID 2: 55 – 80 km

4.9. WEIGHTS

The HANDBIKE BATEC HIBRID models are equipped with standard weights to improve traction. The weights are located on the fork, one on each side. Each one weighs 2.4 kg. To make it easier to transport the HANDBIKE BATEC HIBRID, the weights can be removed.

- To remove the weights:
 1. Hold the weight with one hand, press on the red button and tilt it so the bottom part comes away from the HANDBIKE BATEC HIBRID. (See fig. 31)
 2. Pull the weight out completely. (See fig. 31)
- To insert the weights:
 1. Hold the weight tightly in one hand. Tilt it slightly and insert the two tabs into the slit on the weight mount slot. (See fig. 31)
 2. Push the bottom of the weight inward until it clicks into place. (See fig. 31)

5. INSTALLATION AND ADJUSTMENTS

5.1. INSTALLATION OF THE HANDBIKE BATEC HIBRID ON A WHEELCHAIR

To anchor a HANDBIKE BATEC HIBRID to a wheelchair, you must first install a BATEC DOCKING-BAR (pat. pending). The BATEC DOCKING-BAR can be installed on most wheelchairs on the market, whether fixed or folding frame.

IMPORTANT!

An updated list of wheelchairs compatible with the HANDBIKE BATEC HIBRID is available in the support section of the BATEC MOBILITY website.

The HANDBIKE BATEC HIBRID must be installed on a wheelchair by a qualified professional authorised by BATEC MOBILITY. Contact BATEC MOBILITY to find the authorised installation professional closest to you.

5.2. USER ADJUSTMENTS

Once the HANDBIKE BATEC HIBRID is properly installed on the wheelchair, there are several adjustments that must be made in order to fit the HANDBIKE BATEC HIBRID to the user's level of mobility, type of disability and size.

These adjustments can be done easily with the tools and instructions provided with the HANDBIKE BATEC HIBRID and basic mechanical knowledge.

The following adjustments can be made:

- Adjusting the distance between the HANDBIKE BATEC HIBRID and the wheelchair.
- Adjusting the angle of the stem on the HANDBIKE BATEC HIBRID.
- Adjusting the height of the handlebar on the HANDBIKE BATEC HIBRID.

5.2.1. ADJUST THE DISTANCE BETWEEN THE HANDBIKE BATEC HIBRID AND THE CHAIR

With the HANDBIKE BATEC HIBRID anchored, adjust the distance between the HANDBIKE BATEC HIBRID and the wheelchair so that there is as little space as possible between the user's feet and the stand frame, using the telescopic anchoring bar and the screws indicated. If the user doesn't have any other preference or limitations, we recommend a distance of 8 cm. (See fig. 32)

5.2.2. ADJUST STEM ANGLE

With the HANDBIKE BATEC HIBRID anchored, adjust the angle of the stem on the HANDBIKE BATEC HIBRID in order to fit the handlebar to the user's arm length, using the screws indicated. The upper and lower screws must be set on the same number. If the user doesn't have any other preference or limitations, we recommend position 3. (See fig. 33)

5.2.3. ADJUST STEM HEIGHT

With the HANDBIKE BATEC HIBRID anchored, adjust the stem height on the HANDBIKE BATEC HIBRID with the Quick Release lever. (See fig. 34)

5.2.4. LCD CONSOLE ADJUSTMENTS

1. Switch on the battery to see the INFORMATION SCREEN. (See fig. 35)
2. Press M (mode) button to move between all functions in loop sequence from one function screen to another as CLOCK / ODO (odometer) - RT (partial time) / TT (total time) - AVG (average speed) / MAX (maximum speed) - TRIP 1 / ODO - TRIP 2 / ODO. (See fig. 36)
 - Press R (reset) button for 2 seconds to reset TRIP 2, AVG and MAX from stored values to zero individually.
 - Data of TRIP 1, AVG and RT can be reset at the same time when one of the three data functions is being reset.
 - ODO, CLOCK and TT data cannot be reset.
3. Press M and R buttons simultaneously for 2 seconds to display the SETTING SCREEN 1. (See fig. 37)
4. Press M button to move between all functions in loop sequence from one function screen to another as 12/24 hour, clock, circumference wheel number and Km/h or MPH.

- Press R button to increment the flashing digit by 1 or convert units.
 - Press M button to confirm the digit setting and jump to next digit or next setting screen to be set.
 - The c (circumference number) values are those:
 - HANDBIKE BATEC HIBRID HIBRID 2: 1602
 - Press M for 2 seconds at any setting screen to finish the setting and go to INFORMATION SCREEN
5. On SETTING SCREEN 1 (step 4) pressing M and R buttons simultaneously for 2 seconds will display the SETTING SCREEN 2. **(See fig. 38)**
- Battery voltage value is on the bottom and manufacturer fine adjustment battery value is on top.
6. To return to INFORMATION SCREEN and fix all settings press M button for 2 seconds, if the screen displays SETTING SCREEN 1 press M button for 2 seconds again. **(See fig. 39)**

 **IMPORTANT!**

- Do not change the battery voltage value and fine adjustment battery value.
 - Don't exit SETTING SCREEN 1 by switching off the battery.
 - If the battery is fully charged and the screen display shows the following when switching on, you must follow steps 3, 5 and 6. **(See fig. 40)**
-

6. MAINTENANCE AND REPAIRS

Your HANDBIKE BATEC HIBRID has been carefully designed for optimal performance, but it is important that you carry out certain preventative maintenance tasks in line with the use you give your HANDBIKE BATEC HIBRID. This will greatly reduce the chance of problems and increase the lifespan of your HANDBIKE BATEC HIBRID.

Remember that your main source of information and help is the distributor that delivered your HANDBIKE BATEC HIBRID, who can also respond to any questions regarding adjustments, maintenance and the warranty.

As you are a member of the "BATEC MOBILITY Family", we encourage you to stay in touch with your BATEC HANDBIKE representative in order to make the most of the product.

PREVENTATIVE MAINTENANCE PLAN SCHEDULE

TASKS	Monthly	Yearly
Check tyre pressure. Check general tyre state (tread height, rubbing, cracking, etc.) and change if necessary.	✓	
Clean Batec.	✓	
Tighten screws.	✓	
Check brakes (covers, cables, levers, discs, etc.). Change and/or adjust parts if necessary.	✓	
Check gears (covers, cables, derailleur, etc.). Change and/or adjust if necessary.	✓	
Check lubrication of cables and chains, grease if necessary.	✓	
Check transmission mechanism and coupling, grease if necessary.	✓	
Check spoke tension. Visit an Official Service Centre or Authorised Distributor if necessary.	✓	
Have Batec checked by an Official Service Centre or Authorised Distributor.		✓

Checking tyre pressure:

Make sure the tyre pressure is as recommended for each model and that there are no irregularities on the surface.

Cleaning the HANDBIKE BATEC HIBRID:

Clean the frame with soap and warm water. Use car soap or another non-abrasive cleaning agent. Dry it carefully. Never use a pressure washer to clean it.

Disinfecting the HANDBIKE BATEC HIBRID

To disinfect the HANDBIKE BATEC HIBRID, follow the steps below:

- Use a hydro-alcoholic sanitiser and a cloth.
- Wet the cloth with a generous amount of sanitiser.
- Rub the cloth over any surfaces that come in contact with the user: handle-bars, buttons, display panel and detaching lever.
- Leave to air dry.

→ IMPORTANT!

Sand and salt water can damage some parts of the HANDBIKE BATEC HIBRID after prolonged contact. Clean it after use in these conditions.

Cleaning front and rear wheels:

Make sure the front wheel and rear wheels turn freely and eliminate any dirt.

Checking brakes:

Make sure that when you squeeze the brake levers, both have sufficient contact with the disc for effective braking.

Make sure the brake callipers and disc are properly attached to their respective anchors.

To adjust the brake pads, always do the inside pad first and then the outside one.

Greasing cables and chains:

Apply a drop of grease or light coat of Teflon spray to the cable connecting levers and to chains, after removing the old grease. After applying, remove any excess grease.

Greasing the transmission mechanism:

Apply a drop of grease or light coat of Teflon spray to the transmission mechanism, after removing the old grease. After applying, remove any excess grease.

Having HANDBIKE BATEC HIBRID serviced by an authorised distributor:

The recommended preventive maintenance and any repairs to your BATEC ELECTRIC HANDBIKE must be done by a company authorised by BATEC MOBILITY.

Replacing tyres and inner tubes:

Before installing a new inner tube, check the base of the rim and inside wall of the tyre to remove any foreign bodies and, if necessary, clean well. Check the conditions of the rim tape, particularly around the valve hole. Only strong, properly placed rim tape can provide long-term protection against damage to the inner tube caused by burrs and sharp spoke heads.

Before inflating the inner tube, check the whole tube to make sure it hasn't got stuck between the tyre and the rim. Carefully pull on the valve and extract it for optimal installation.

Reusing the BATEC ELECTRIC HANDBIKE

The BATEC ELECTRIC HANDBIKE can be reused. If you would like to reuse it, please contact the distributor where you purchased it and they will take care of

sending the unit to be reused to the BATEC MOBILITY facilities. The unit to be reused will be checked, given a tune-up and re-checked.

CAREFUL!

After installing the new inner tube, check tyre pressure. It is essential to keep pressure at specified levels and to make sure tyres are in proper conditions in order to ensure your personal safety and keep your HANDBIKE BATEC HIBRID in proper working order.

IMPORTANT!

As the necessary maintenance varies between users, check with your BATEC MOBILITY representative to establish a periodic maintenance schedule that fits your specific use.

IMPORTANT!

Always do a safety check of your HANDBIKE BATEC HIBRID before using. Improper maintenance may invalidate the warranty.

7. LABEL

All BATEC ELECTRIC HANDBIKES have a label on the bottom of the frame bar with the following information: **(See fig. 41)**

SYMBOLS:

	Manufacturer
SN	Serial number
REF	Catalogue number
	This product complies with the requirements of Regulation (EU) 2017/745 on medical devices
	Date manufactured
	See user instructions
MD	With the text "Medical Device"
	Check warnings (battery)
	EN 12184:2014 – Wheelchair not intended for use as a seat in a motor vehicle
	ISO 7000 – Maximum weight allowed

When ordering spare parts or lodging claims, please provide the information on the label.

8. WARRANTY

8.1. WARRANTY PERIOD

BATEC MOBILITY guarantees all hard and structural parts for a **period of three (3) years** from the date of delivery in the case of material defects or labour for the original registered owner.

BATEC MOBILITY guarantees the BATTERY for a **period of one (1) year** from the date of delivery in the case of material defects or labour for the original registered owner. The BATTERY is considered a fungible part.

8.2. WARRANTY CONDITIONS

- Fungibles (except for the BATTERY, which has a one-year warranty) are excluded from the warranty, except in the case of premature wear on said parts caused by original manufacturing defects. These elements include tyres, inner tubes, handles, cables for brakes, gear shifter and anchoring, stand frame wheels and similar parts.
- If you have any questions regarding the warranty, BATEC MOBILITY recommends first contacting your authorised BATEC MOBILITY distributor, who will explain the solutions available for the problem.
- BATEC MOBILITY shall not be held responsible for any defects that may arise as a result of improper use or storage of the HANDBIKE BATEC HIBRID. If BATEC MOBILITY considers that a part has not been properly installed, used or stored, the warranty will be invalidated. Some examples are as follows:
 - The product or part hasn't been maintained according to BATEC MOBILITY recommendations, as detailed in the user instructions and maintenance instructions.
 - Non-original accessories have been used.
 - The HANDBIKE BATEC HIBRID or its parts have been damaged due to negligence, accident or improper use.
 - The HANDBIKE BATEC HIBRID or its parts have been modified in a way that is not in line with BATEC MOBILITY specifications or have undergone repairs without notifying the authorised distributor or technical service provider.
- BATEC MOBILITY reserves the right, with any claim, to require the claimant to send all parts to BATEC MOBILITY for replacement or repair at their own cost. If any of the parts are found to be defective, BATEC MOBILITY will determine whether to replace, repair or maintain the parts.

- This warranty doesn't include transport, shipping or labour costs for removing and reinstalling the parts. BATEC MOBILITY will charge the claimant for any costs incurred if the parts are found not to be defective or the damage has been caused by events not covered under this warranty.
- Defective parts will be repaired or replaced exclusively by BATEC MOBILITY or a duly qualified official BATEC MOBILITY service provider.
- BATEC MOBILITY doesn't ensure any of its products or parts will be available on the market or suitable for any particular purpose and provides no guarantee, express or implied, beyond the attached limited warranty.
- Replacement parts installed during the original warranty period shall have an additional 2-year warranty. For the BATTERY, as it is considered a fungible part, the replacement unit shall be under warranty for the remaining warranty period on the original BATTERY, up to 1 year from the date the HANDBIKE BATEC HIBRID was delivered.
- This warranty is non-transferable: This warranty is only valid for the original registered user, who must provide the original receipt as proof of purchase.
- BATEC MOBILITY shall not be held responsible for any products not purchased at the recommended retail price.
- Except where prohibited by law, this limited warranty constitutes the full agreement between the owner and BATEC MOBILITY regarding any sort of liability that may arise concerning any parts on the HANDBIKE BATEC HIBRID products.

 **IMPORTANT!**

- **BATEC MOBILITY** and the competent authority in the Member State of residence **should be notified of any serious incidents** related to the use of the product.
-

8.3. WARRANTY CARD

With the rest of the documentation for your HANDBIKE BATEC HIBRID, you will find a Warranty Card with all the details regarding your HANDBIKE BATEC HIBRID. You can use this card to provide all the details of your HANDBIKE BATEC HIBRID if a situation arises that requires the warranty.

9. REVISIONS LIST

Version	Summary / justification of changes
1.0	To adapt to the requirements of Regulation (EU) 2017/745
2.0	Single manual with all languages

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